



**URGENT MEDICAL DEVICE RECALL**

HeartMate™ Mobile Power Unit (MPU)  
 Used with HeartMate 3™ LVAS and HeartMate II® LVAS  
 (Model Number: 107754)

Heart Failure Division  
 Abbott Medical  
 6305 Stoneridge Dr.  
 Pleasanton, CA 94588

March 2025

Dear Physician or Healthcare Professional,

The purpose of this letter is to advise you that Abbott is initiating a voluntary field action for the HeartMate™ Mobile Power Unit (MPU). Abbott has received customer complaints in which the MPU exhibited sudden, unexpected performance issues such as not turning on, unprompted shut down, or suddenly turning off and restarting, with the System Controller indicating a Yellow Wrench alarm or “No External Power” alarm. These issues have been linked to an electrical component in certain MPUs distributed between April 2024 and February 2025.

Impact and Associated Risk

As of February 28, 2025, the complaint rate of MPU performance interruptions within the affected device population is approximately 4%. The reported impacts on patients included anxiety and inconvenience. Although not having been reported, this issue could also potentially lead to serious adverse health consequences such as hemodynamic compromise, thromboembolism, or death when power to the HeartMate left ventricular device is interrupted.

Root Cause

Investigation of the returned MPUs identified that an electrical component in the power supply does not meet Abbott quality standards and specifications. Not all MPUs from the identified manufacturing time frame contain the nonconforming electrical component. A list of the affected MPUs is provided in Appendix A.

How to Recognize the Issue and React

If an impacted MPU experiences a loss of power, the visual/audio alarms, as shown in Table 1, will deploy. When this happens, the Backup Battery in the System Controller can support the pump for up to 15 minutes. It is critical to switch from the MPU to the 14V rechargeable batteries immediately. If the batteries are not connected to the System Controller within 15 minutes, the System Controller Backup Battery will deplete, causing pump power loss.



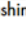
Alarm Symbol	Meaning	What You Should Do
 <p><b>Advisory Alarm</b> Yellow wrench light with beeping audio tone</p>	Internal malfunction detected within the MPU.	
 <ul style="list-style-type: none"> <li>• Flashing Red Battery (  ) on the user interface.</li> <li>• “Connect Power Immediately” and Backup Battery graphic alternate on the screen.</li> <li>• Yellow light near the black power cable connector is flashing.</li> <li>• Yellow light near the white power cable connector is flashing.</li> <li>• Alarm tone: Constant tone.</li> </ul>	System Controller is not receiving external power.	Promptly switch to two fully charged HeartMate™ 14 Volt Lithium-Ion batteries.

Table 1. Mobile Power Unit Alarm

### Hospital User Action Requested

Review the enclosed Appendix A to identify potentially impacted MPUs that are either on your shelves, used in your clinic, or already assigned to patients.

- If patients **are** able to be identified using the serial number list (Appendix A), please send those patients the patient letter (Appendix B).
- If patients **are not** able to be identified using the serial number list (Appendix A), please send the patient letter (Appendix B) to all patients that have received an MPU from April 2024 to March 2025.
- For all other patients with MPUs not listed in Appendix A, they need not be notified. If the MPU is not listed in Appendix A, this device is not affected and can continue to be used.

Depending on the status of the impacted MPUs at your facility or with your patients, Abbott recommends the following:

#### *For units that patients are currently using and have experienced MPU power issues:*

- Follow the instructions provided in the Instructions for Use (IFU): If there is an MPU power failure, transfer the patient from the MPU to the 14V rechargeable batteries within 15 minutes. The Backup Battery in the System Controller will temporarily power the pump during a power source switch. Do not rely only on the System Controller's Backup Battery as a power source during power failure, as it will only power the pump for up to 15 minutes.
  - Note: MPU will continue to echo the controller alarms as long as the internal AA batteries in the MPU do not need replacement.
- Do not continue to use the MPU and immediately contact Abbott Technical Support at 1-800-456-1477 (Business hours: 8 AM EST to 7 PM EST) for return and replacement.

#### *For units currently used by patients, but **not** experiencing MPU power issues:*

- Provide the patient with the attached patient letter (Appendix B).
- As a precaution educate the patient about the issue and
  - Ensure the patient's 14V rechargeable batteries are ready and available for use at any time.
  - Prompt the patient to replace AA batteries inside the MPU immediately if the Yellow Mobile Power Unit Battery Indicator alarm is active. The internal AA batteries ensure that the MPU echoes the System Controller Alarms.
- Abbott will replace these with new units starting June 2025, even if these units continue to demonstrate expected performance.

#### *For impacted MPU units that are currently in your clinic and have not yet been provided to a patient for use:*

- Immediate return and replace units not distributed, not in use, on shelves, or that are unopened.
- Contact Abbott Customer Support at 1-612-444-3013, Option 2 (Business Hours: 9 AM EST to 6:30 PM EST) to arrange for product return and inventory replacement.

Please distribute this notice to those who need to be aware within your institution and to any organization to which these devices may have been transferred. A device lookup tool with the impacted MPU devices will be available on the Abbott Product Advisories website at <https://www.cardiovascular.abbott/us/en/hcp/products/heart-failure/left-ventricular-assist-devices/heartmate-3/indications-safety-warnings/advisories.html> by March 24<sup>th</sup>, 2025. Additionally, please complete the attached acknowledgment form and return it to Abbott.

Abbott Action

Abbott is taking the following actions:

- Notifying the clinic of patients with the impacted MPUs, and any impacted in-clinic inventory (Appendix A).
- Notifying patients with impacted MPUs through the clinics using the attached patient letter (Appendix B).
- Immediately replacing MPU devices that are demonstrating unexpected performance, and any impacted clinic inventory. Replacement of all remaining impacted MPU devices will begin in June 2025 or earlier when sufficient inventory is available. Your local Abbott representative will work with you to ensure a smooth transition.

Abbott has notified the United States Food & Drug Administration (FDA) about these issues. Should you have any questions about this notice, please contact your Abbott representative or Abbott Technical Support at 1-800-456-1477 (Business hours: 8 AM EST to 7 PM EST).

Adverse reactions or quality problems experienced with the use of this product may be reported to the FDA's MedWatch Adverse Event Reporting program either online, by regular mail or by fax.

- Complete the voluntary Form FDA 3500 online at <https://www.fda.gov/safety/medical-product-safety-information/medwatch-forms-fda-safety-reporting>.
- Download the form from FDA.gov or call 1-800-332-1088 to request a reporting form, then complete and return it to the address on the pre-addressed form or submit it by fax to 1-800-FDA-0178.

Please know that Abbott is committed to providing the highest quality products and support, and we thank you for your partnership in assisting us with this process.

Sincerely,



Carolyn Tabion  
Divisional Vice President, Quality  
Abbott Heart Failure

Enclosed:

- Appendix A – Reference Pictures and Listing of Potentially Impacted MPUs and Patients
- Appendix B – Patient Letter
- Acknowledgment Form




**Appendix A – Reference Pictures and Listing of Potentially Impacted MPUs**

**Images of Location of MPU Serial Number**

Product Packaging Label

Device Label

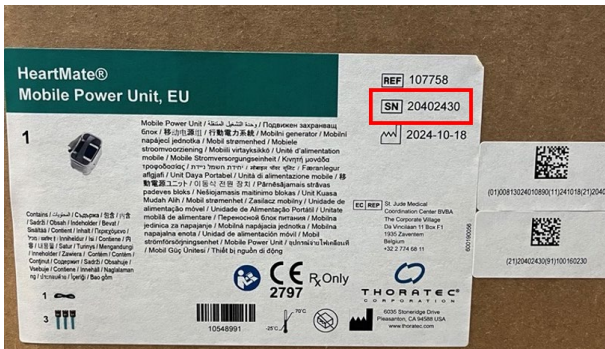

Pictures are for reference only of the Serial Number location (pictures are of EU and US product label examples).

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20226263
107754	20226264
107754	20226265
107754	20226266
107754	20226267
107754	20226268
107754	20226269
107754	20226270
107754	20226271
107754	20226272
107754	20226273
107754	20226274
107754	20226275
107754	20226276
107754	20226277
107754	20226278
107754	20226279
107754	20226280
107754	20226281
107754	20226282
107754	20226283
107754	20226284
107754	20226285
107754	20226286
107754	20226287
107754	20226288
107754	20226289
107754	20226290
107754	20226291
107754	20226292
107754	20240622
107754	20240623
107754	20240624
107754	20240625
107754	20240626
107754	20240627
107754	20240628
107754	20240629
107754	20240630
107754	20240631
107754	20240632
107754	20240633
107754	20240634
107754	20240635
107754	20240636
107754	20240637

MPU Model Number	MPU Serial Number
107754	20240638
107754	20240639
107754	20240640
107754	20240641
107754	20240642
107754	20240643
107754	20240644
107754	20240645
107754	20240646
107754	20240647
107754	20240648
107754	20240649
107754	20240650
107754	20240651
107754	20240652
107754	20240653
107754	20240654
107754	20240655
107754	20240656
107754	20240657
107754	20240658
107754	20240659
107754	20240660
107754	20240661
107754	20240662
107754	20240663
107754	20240664
107754	20240665
107754	20240666
107754	20240667
107754	20240668
107754	20240669
107754	20240670
107754	20240671
107754	20240672
107754	20240673
107754	20240674
107754	20240675
107754	20240676
107754	20240677
107754	20240678
107754	20240679
107754	20240680
107754	20240681
107754	20240682
107754	20240683

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20240684
107754	20240685
107754	20240686
107754	20240687
107754	20240688
107754	20240689
107754	20240690
107754	20240691
107754	20254123
107754	20254124
107754	20254125
107754	20254126
107754	20254127
107754	20254128
107754	20254129
107754	20254130
107754	20254131
107754	20254132
107754	20254133
107754	20254134
107754	20254135
107754	20254136
107754	20254137
107754	20254138
107754	20254139
107754	20254140
107754	20254141
107754	20254142
107754	20254143
107754	20254144
107754	20254145
107754	20254146
107754	20254147
107754	20254148
107754	20254149
107754	20254150
107754	20254151
107754	20254152
107754	20254153
107754	20254154
107754	20254155
107754	20254156
107754	20254157
107754	20254158
107754	20254159
107754	20254160

MPU Model Number	MPU Serial Number
107754	20254161
107754	20254162
107754	20254166
107754	20254167
107754	20254168
107754	20254169
107754	20254170
107754	20254171
107754	20254172
107754	20254173
107754	20254174
107754	20254175
107754	20254176
107754	20254177
107754	20254178
107754	20254179
107754	20254180
107754	20254181
107754	20254182
107754	20265454
107754	20265455
107754	20265456
107754	20265457
107754	20265458
107754	20265459
107754	20265460
107754	20265461
107754	20265462
107754	20265463
107754	20265464
107754	20265465
107754	20265466
107754	20265467
107754	20265468
107754	20265469
107754	20265470
107754	20265471
107754	20265472
107754	20265473
107754	20265474
107754	20265475
107754	20265476
107754	20265477
107754	20265478
107754	20265479
107754	20265480

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20265481
107754	20265483
107754	20265484
107754	20265485
107754	20265486
107754	20265487
107754	20265488
107754	20265489
107754	20265490
107754	20265491
107754	20265492
107754	20265493
107754	20265494
107754	20265495
107754	20265496
107754	20265497
107754	20265498
107754	20265499
107754	20265500
107754	20265501
107754	20265502
107754	20265503
107754	20265504
107754	20265505
107754	20265506
107754	20265507
107754	20265508
107754	20265509
107754	20265510
107754	20265511
107754	20265512
107754	20265513
107754	20265514
107754	20265515
107754	20265516
107754	20265517
107754	20265518
107754	20265519
107754	20265520
107754	20265521
107754	20265522
107754	20265523
107754	20265524
107754	20265525
107754	20265526
107754	20265527

MPU Model Number	MPU Serial Number
107754	20265528
107754	20265529
107754	20265530
107754	20265531
107754	20265532
107754	20265533
107754	20265534
107754	20265535
107754	20265536
107754	20265537
107754	20265538
107754	20265539
107754	20265540
107754	20265541
107754	20265542
107754	20265543
107754	20265544
107754	20265545
107754	20265546
107754	20265547
107754	20265548
107754	20265549
107754	20265550
107754	20265551
107754	20265552
107754	20265553
107754	20265554
107754	20265555
107754	20265556
107754	20265557
107754	20265558
107754	20265559
107754	20265560
107754	20265561
107754	20265562
107754	20265563
107754	20265564
107754	20265565
107754	20265566
107754	20265567
107754	20265568
107754	20265569
107754	20265570
107754	20265571
107754	20265572
107754	20265573

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20265574
107754	20265576
107754	20265577
107754	20265578
107754	20265579
107754	20265580
107754	20265581
107754	20265582
107754	20265583
107754	20265584
107754	20265585
107754	20265586
107754	20265587
107754	20265588
107754	20265589
107754	20265590
107754	20265591
107754	20265592
107754	20265593
107754	20265594
107754	20265595
107754	20265596
107754	20265597
107754	20265598
107754	20265599
107754	20265600
107754	20265601
107754	20265602
107754	20265603
107754	20265604
107754	20265605
107754	20265606
107754	20265607
107754	20265608
107754	20265609
107754	20265610
107754	20265611
107754	20265612
107754	20265613
107754	20265614
107754	20265615
107754	20265616
107754	20265617
107754	20265618
107754	20265619
107754	20265620

MPU Model Number	MPU Serial Number
107754	20265621
107754	20265622
107754	20265623
107754	20265624
107754	20265625
107754	20265626
107754	20265627
107754	20265628
107754	20265629
107754	20265630
107754	20265631
107754	20265632
107754	20265633
107754	20265634
107754	20265635
107754	20265636
107754	20265638
107754	20265639
107754	20265641
107754	20265642
107754	20265643
107754	20265644
107754	20273242
107754	20273243
107754	20273244
107754	20273245
107754	20273246
107754	20273247
107754	20273248
107754	20273249
107754	20273250
107754	20273251
107754	20273252
107754	20273253
107754	20273254
107754	20273255
107754	20273256
107754	20273257
107754	20273258
107754	20273259
107754	20273260
107754	20273261
107754	20273262
107754	20273263
107754	20273264
107754	20273265



## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20273266
107754	20273267
107754	20273268
107754	20273269
107754	20273270
107754	20273271
107754	20285420
107754	20285421
107754	20285422
107754	20285423
107754	20285424
107754	20285425
107754	20285426
107754	20285427
107754	20285428
107754	20285429
107754	20285430
107754	20285431
107754	20285432
107754	20285433
107754	20285434
107754	20285435
107754	20285436
107754	20285437
107754	20285438
107754	20285439
107754	20285461
107754	20285463
107754	20285464
107754	20285465
107754	20285467
107754	20285468
107754	20285469
107754	20285470
107754	20285471
107754	20285472
107754	20285473
107754	20285475
107754	20285476
107754	20285477
107754	20285478
107754	20285479
107754	20285480
107754	20285481
107754	20285483
107754	20285484

MPU Model Number	MPU Serial Number
107754	20285485
107754	20285486
107754	20285487
107754	20285488
107754	20285489
107754	20285490
107754	20285491
107754	20285492
107754	20285493
107754	20285494
107754	20285495
107754	20285496
107754	20285497
107754	20285498
107754	20285499
107754	20285500
107754	20285501
107754	20285502
107754	20285503
107754	20285569
107754	20285570
107754	20285571
107754	20285572
107754	20285573
107754	20285574
107754	20285575
107754	20285576
107754	20285577
107754	20285578
107754	20285589
107754	20285590
107754	20285591
107754	20285592
107754	20285593
107754	20285594
107754	20285595
107754	20285596
107754	20285597
107754	20285598
107754	20285599
107754	20285600
107754	20285601
107754	20285602
107754	20285603
107754	20285604
107754	20285605

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20285606
107754	20285608
107754	20285609
107754	20291598
107754	20291599
107754	20291600
107754	20291601
107754	20291602
107754	20291603
107754	20291604
107754	20291605
107754	20291606
107754	20291607
107754	20291608
107754	20291609
107754	20291610
107754	20291611
107754	20291612
107754	20291613
107754	20291614
107754	20291615
107754	20291616
107754	20291617
107754	20291618
107754	20291619
107754	20291620
107754	20291621
107754	20291622
107754	20291623
107754	20291624
107754	20291625
107754	20291627
107754	20291628
107754	20291629
107754	20291630
107754	20291631
107754	20291632
107754	20291633
107754	20291634
107754	20291635
107754	20291636
107754	20291637
107754	20293120
107754	20293121
107754	20293122
107754	20293123

MPU Model Number	MPU Serial Number
107754	20293124
107754	20293125
107754	20293126
107754	20293127
107754	20293128
107754	20293129
107754	20293130
107754	20293131
107754	20293132
107754	20293133
107754	20293134
107754	20293135
107754	20293136
107754	20293137
107754	20293138
107754	20293139
107754	20293140
107754	20293141
107754	20293142
107754	20293143
107754	20293144
107754	20293145
107754	20293146
107754	20293147
107754	20293148
107754	20293149
107754	20293150
107754	20293151
107754	20293152
107754	20293153
107754	20293154
107754	20293155
107754	20293156
107754	20293157
107754	20293158
107754	20293159
107754	20296547
107754	20296548
107754	20296549
107754	20296550
107754	20296551
107754	20296552
107754	20296553
107754	20296555
107754	20296556
107754	20296557

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20296559
107754	20296560
107754	20296561
107754	20296563
107754	20296564
107754	20296565
107754	20296566
107754	20296567
107754	20296568
107754	20296569
107754	20296571
107754	20296572
107754	20296573
107754	20296574
107754	20296575
107754	20296576
107754	20296577
107754	20297159
107754	20297160
107754	20297161
107754	20297162
107754	20297163
107754	20297164
107754	20297165
107754	20297166
107754	20297167
107754	20297168
107754	20302246
107754	20302247
107754	20302248
107754	20302249
107754	20302250
107754	20302251
107754	20302252
107754	20302254
107754	20302255
107754	20302256
107754	20302257
107754	20302258
107754	20302260
107754	20302261
107754	20302262
107754	20302264
107754	20302265
107754	20302266
107754	20302267

MPU Model Number	MPU Serial Number
107754	20302268
107754	20302269
107754	20302270
107754	20302272
107754	20302273
107754	20302274
107754	20302275
107754	20302276
107754	20302277
107754	20302278
107754	20302279
107754	20302280
107754	20302281
107754	20302282
107754	20302284
107754	20302285
107754	20302287
107754	20302288
107754	20302289
107754	20302290
107754	20302291
107754	20302292
107754	20302293
107754	20302294
107754	20302295
107754	20302297
107754	20302298
107754	20302299
107754	20302300
107754	20302301
107754	20302302
107754	20302303
107754	20302304
107754	20302305
107754	20302307
107754	20302308
107754	20302309
107754	20302310
107754	20302311
107754	20302312
107754	20302313
107754	20302314
107754	20302315
107754	20302316
107754	20302317
107754	20302318

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20302319
107754	20302320
107754	20302321
107754	20302322
107754	20302323
107754	20302324
107754	20302325
107754	20302326
107754	20302327
107754	20302328
107754	20302329
107754	20302330
107754	20302331
107754	20302332
107754	20302333
107754	20302334
107754	20302335
107754	20302336
107754	20302337
107754	20302338
107754	20302339
107754	20302340
107754	20302341
107754	20302342
107754	20302343
107754	20302344
107754	20302345
107754	20302346
107754	20302347
107754	20302348
107754	20302349
107754	20302350
107754	20302351
107754	20302352
107754	20302353
107754	20302354
107754	20302355
107754	20307844
107754	20307845
107754	20307846
107754	20307847
107754	20307848
107754	20307849
107754	20307850
107754	20307851
107754	20307852

MPU Model Number	MPU Serial Number
107754	20307853
107754	20307854
107754	20307855
107754	20307856
107754	20307857
107754	20307858
107754	20307859
107754	20307860
107754	20307861
107754	20307862
107754	20307863
107754	20307864
107754	20307865
107754	20307866
107754	20307867
107754	20307868
107754	20307869
107754	20307870
107754	20307872
107754	20307873
107754	20307874
107754	20307875
107754	20307876
107754	20307877
107754	20307878
107754	20307879
107754	20307880
107754	20307881
107754	20307882
107754	20307883
107754	20307884
107754	20307885
107754	20307886
107754	20307887
107754	20307888
107754	20307889
107754	20307890
107754	20307891
107754	20307892
107754	20307893
107754	20307894
107754	20307895
107754	20307897
107754	20307898
107754	20307899
107754	20307900

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20307901
107754	20307902
107754	20307903
107754	20307904
107754	20307905
107754	20307906
107754	20307907
107754	20307909
107754	20307910
107754	20307911
107754	20307912
107754	20307913
107754	20307914
107754	20307915
107754	20307917
107754	20307918
107754	20307919
107754	20307920
107754	20307921
107754	20307923
107754	20307924
107754	20307926
107754	20307927
107754	20307928
107754	20307929
107754	20307930
107754	20307931
107754	20307932
107754	20307933
107754	20307934
107754	20307935
107754	20307936
107754	20307937
107754	20307938
107754	20307939
107754	20307940
107754	20307941
107754	20307942
107754	20307944
107754	20307945
107754	20307946
107754	20307948
107754	20307949
107754	20307950
107754	20307951
107754	20307952

MPU Model Number	MPU Serial Number
107754	20307953
107754	20307954
107754	20307955
107754	20307957
107754	20307958
107754	20307959
107754	20307960
107754	20307961
107754	20307962
107754	20307963
107754	20307964
107754	20307965
107754	20307966
107754	20307967
107754	20307968
107754	20307969
107754	20307970
107754	20307971
107754	20307972
107754	20307973
107754	20307974
107754	20307975
107754	20307976
107754	20307977
107754	20307978
107754	20307979
107754	20307980
107754	20307981
107754	20307982
107754	20307983
107754	20307984
107754	20307985
107754	20307986
107754	20307987
107754	20307988
107754	20307990
107754	20307991
107754	20307992
107754	20307993
107754	20309412
107754	20309413
107754	20309414
107754	20309415
107754	20309417
107754	20309418
107754	20309419

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20309420
107754	20309421
107754	20309422
107754	20309423
107754	20309424
107754	20309425
107754	20309427
107754	20309428
107754	20309429
107754	20309430
107754	20309432
107754	20309433
107754	20309434
107754	20309435
107754	20309436
107754	20309438
107754	20309439
107754	20309440
107754	20309441
107754	20309442
107754	20309443
107754	20309444
107754	20309445
107754	20309446
107754	20309448
107754	20309449
107754	20309450
107754	20309451
107754	20309452
107754	20309453
107754	20309454
107754	20309455
107754	20309456
107754	20309457
107754	20309458
107754	20309459
107754	20309460
107754	20309461
107754	20309462
107754	20309463
107754	20309464
107754	20309465
107754	20309466
107754	20309467
107754	20309468
107754	20309469

MPU Model Number	MPU Serial Number
107754	20309470
107754	20309471
107754	20309472
107754	20309473
107754	20309474
107754	20309475
107754	20309476
107754	20309477
107754	20309478
107754	20309479
107754	20309480
107754	20309481
107754	20309482
107754	20309483
107754	20309484
107754	20309485
107754	20309486
107754	20309487
107754	20309488
107754	20309489
107754	20309490
107754	20309491
107754	20309492
107754	20309493
107754	20309494
107754	20309495
107754	20309496
107754	20309497
107754	20309499
107754	20309500
107754	20309501
107754	20309502
107754	20309503
107754	20309504
107754	20309505
107754	20309506
107754	20309507
107754	20309508
107754	20309509
107754	20309510
107754	20309511
107754	20309512
107754	20309513
107754	20309514
107754	20309516
107754	20309517

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20309518
107754	20309519
107754	20309520
107754	20309521
107754	20309522
107754	20309523
107754	20309524
107754	20309525
107754	20309527
107754	20309528
107754	20309529
107754	20309530
107754	20309531
107754	20309582
107754	20309583
107754	20309584
107754	20309585
107754	20309586
107754	20309587
107754	20309588
107754	20309589
107754	20309590
107754	20309591
107754	20309592
107754	20309593
107754	20309594
107754	20309595
107754	20309596
107754	20309597
107754	20309598
107754	20309599
107754	20309600
107754	20309601
107754	20309752
107754	20309753
107754	20309754
107754	20309755
107754	20309756
107754	20309757
107754	20309758
107754	20309759
107754	20309760
107754	20309761
107754	20314564
107754	20314565
107754	20314566

MPU Model Number	MPU Serial Number
107754	20314567
107754	20314568
107754	20314569
107754	20314570
107754	20314571
107754	20314572
107754	20314573
107754	20314574
107754	20314575
107754	20314576
107754	20314578
107754	20314579
107754	20314580
107754	20314581
107754	20314582
107754	20314583
107754	20314584
107754	20314585
107754	20314586
107754	20314587
107754	20314588
107754	20314590
107754	20314591
107754	20314592
107754	20314593
107754	20314594
107754	20314595
107754	20314596
107754	20314597
107754	20314598
107754	20314599
107754	20314600
107754	20314601
107754	20314602
107754	20314603
107754	20314604
107754	20314605
107754	20314606
107754	20314607
107754	20314608
107754	20314609
107754	20314610
107754	20314611
107754	20314612
107754	20314613
107754	20314614

## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20314615
107754	20314616
107754	20314617
107754	20314618
107754	20314619
107754	20314620
107754	20314621
107754	20314622
107754	20314623
107754	20314624
107754	20314625
107754	20314626
107754	20314627
107754	20314628
107754	20314629
107754	20314630
107754	20314631
107754	20314632
107754	20314633
107754	20314634
107754	20314635
107754	20314636
107754	20314637
107754	20314638
107754	20314639
107754	20314640
107754	20314641
107754	20314642
107754	20314643
107754	20314644
107754	20314645
107754	20314646
107754	20314647
107754	20314648
107754	20314649
107754	20314650
107754	20314651
107754	20314653
107754	20314654
107754	20314655
107754	20314656
107754	20314657
107754	20314658
107754	20314659
107754	20314660
107754	20314661

MPU Model Number	MPU Serial Number
107754	20314662
107754	20314663
107754	20314665
107754	20314666
107754	20314667
107754	20314668
107754	20314669
107754	20314670
107754	20314671
107754	20314672
107754	20314673
107754	20314674
107754	20314675
107754	20314676
107754	20314677
107754	20314678
107754	20314679
107754	20314680
107754	20314681
107754	20314682
107754	20314683
107754	20314684
107754	20314685
107754	20314686
107754	20314687
107754	20314688
107754	20314689
107754	20314690
107754	20314691
107754	20314692
107754	20314693
107754	20314694
107754	20314695
107754	20314696
107754	20314697
107754	20314698
107754	20314699
107754	20314700
107754	20314701
107754	20314702
107754	20314703
107754	20326945
107754	20326946
107754	20326947
107754	20326948
107754	20326949



## Listing of Potentially Impacted MPUs

Model 107754, Serial Number Range: 20226263 - 20403981

MPU Model Number	MPU Serial Number
107754	20326950
107754	20326951
107754	20326952
107754	20326953
107754	20326954
107754	20326955
107754	20326956
107754	20326957
107754	20326958
107754	20326959
107754	20326960
107754	20326961
107754	20326962
107754	20326963
107754	20326964
107754	20326975
107754	20326976
107754	20326977
107754	20326978
107754	20326979
107754	20326980
107754	20326981
107754	20326982
107754	20326983
107754	20326984
107754	20326985
107754	20326986
107754	20326987
107754	20326988
107754	20326989
107754	20326990
107754	20326991
107754	20326992
107754	20326993
107754	20326994
107754	20326995
107754	20326996
107754	20326997
107754	20326998
107754	20326999
107754	20327000
107754	20327001
107754	20327002
107754	20327003
107754	20327004
107754	20403978

MPU Model Number	MPU Serial Number
107754	20403979
107754	20403980
107754	20403981

## Listing of Potentially Impacted MPUs

Model 107758, Serial Number Range: 20226183 - 20326942

MPU Model Number	MPU Serial Number
107758	20226183
107758	20226184
107758	20226185
107758	20226186
107758	20226187
107758	20226188
107758	20226189
107758	20226190
107758	20226191
107758	20226192
107758	20226193
107758	20226194
107758	20226195
107758	20226196
107758	20226197
107758	20226198
107758	20226199
107758	20226200
107758	20226201
107758	20226202
107758	20242038
107758	20242039
107758	20242040
107758	20242041
107758	20242042
107758	20242043
107758	20242044
107758	20242045
107758	20242046
107758	20242047
107758	20254063
107758	20254064
107758	20254065
107758	20254066
107758	20254067
107758	20254068
107758	20254069
107758	20254070
107758	20254071
107758	20254072
107758	20254083
107758	20254084
107758	20254085
107758	20254086
107758	20254087
107758	20254088

MPU Model Number	MPU Serial Number
107758	20254090
107758	20254091
107758	20254092
107758	20254093
107758	20254094
107758	20254095
107758	20254096
107758	20254097
107758	20254098
107758	20254099
107758	20254100
107758	20254101
107758	20254102
107758	20254103
107758	20254104
107758	20254105
107758	20254106
107758	20254107
107758	20254108
107758	20254109
107758	20254110
107758	20254111
107758	20254112
107758	20265434
107758	20265435
107758	20265436
107758	20265437
107758	20265438
107758	20265439
107758	20265440
107758	20265441
107758	20265442
107758	20265443
107758	20265444
107758	20265445
107758	20265446
107758	20265447
107758	20265448
107758	20265449
107758	20265450
107758	20265451
107758	20265452
107758	20265453
107758	20273202
107758	20273203
107758	20273204

## Listing of Potentially Impacted MPUs

Model 107758, Serial Number Range: 20226183 - 20326942

MPU Model Number	MPU Serial Number
107758	20273205
107758	20273206
107758	20273207
107758	20273208
107758	20273209
107758	20273210
107758	20273211
107758	20277226
107758	20277227
107758	20277228
107758	20277229
107758	20277230
107758	20277231
107758	20277232
107758	20277233
107758	20277234
107758	20277235
107758	20277236
107758	20277237
107758	20277238
107758	20277240
107758	20277241
107758	20277242
107758	20277243
107758	20277244
107758	20277245
107758	20277246
107758	20277247
107758	20277248
107758	20277249
107758	20277250
107758	20277251
107758	20277252
107758	20277253
107758	20277254
107758	20277255
107758	20277256
107758	20277257
107758	20277258
107758	20277259
107758	20277260
107758	20277261
107758	20277262
107758	20277263
107758	20277264
107758	20277265

MPU Model Number	MPU Serial Number
107758	20277266
107758	20277267
107758	20277268
107758	20277269
107758	20277270
107758	20277271
107758	20277272
107758	20277273
107758	20277274
107758	20285440
107758	20285441
107758	20285442
107758	20285443
107758	20285444
107758	20285445
107758	20285446
107758	20285447
107758	20285448
107758	20285449
107758	20285450
107758	20285451
107758	20285452
107758	20285453
107758	20285454
107758	20285455
107758	20285456
107758	20285457
107758	20285459
107758	20285460
107758	20285504
107758	20285505
107758	20285506
107758	20285507
107758	20285508
107758	20285509
107758	20285510
107758	20285511
107758	20285512
107758	20285513
107758	20285549
107758	20285551
107758	20285552
107758	20285553
107758	20285554
107758	20285555
107758	20285556

## Listing of Potentially Impacted MPUs

Model 107758, Serial Number Range: 20226183 - 20326942

MPU Model Number	MPU Serial Number
107758	20285557
107758	20285558
107758	20285559
107758	20285560
107758	20285561
107758	20285562
107758	20285563
107758	20285564
107758	20285565
107758	20285566
107758	20285567
107758	20285568
107758	20296506
107758	20296507
107758	20296508
107758	20296509
107758	20296510
107758	20296511
107758	20296512
107758	20296513
107758	20296514
107758	20296515
107758	20296516
107758	20296517
107758	20296518
107758	20296519
107758	20296520
107758	20296521
107758	20296522
107758	20296523
107758	20296524
107758	20296526
107758	20296527
107758	20296528
107758	20296529
107758	20296530
107758	20296531
107758	20296532
107758	20296533
107758	20296535
107758	20296536
107758	20296537
107758	20296538
107758	20296539
107758	20296540
107758	20296541

MPU Model Number	MPU Serial Number
107758	20296543
107758	20296544
107758	20296545
107758	20297139
107758	20297140
107758	20297141
107758	20297142
107758	20297143
107758	20297144
107758	20297145
107758	20297146
107758	20297148
107758	20297149
107758	20297150
107758	20297151
107758	20297152
107758	20297153
107758	20297154
107758	20297155
107758	20297156
107758	20297157
107758	20297158
107758	20297169
107758	20297170
107758	20297171
107758	20297172
107758	20297173
107758	20297174
107758	20297175
107758	20297177
107758	20297178
107758	20297179
107758	20297180
107758	20297182
107758	20297184
107758	20297185
107758	20297186
107758	20297187
107758	20300896
107758	20300898
107758	20300899
107758	20300900
107758	20300901
107758	20300902
107758	20300903
107758	20300904

## Listing of Potentially Impacted MPUs

Model 107758, Serial Number Range: 20226183 - 20326942

MPU Model Number	MPU Serial Number
107758	20300905
107758	20300906
107758	20300907
107758	20300908
107758	20300909
107758	20300910
107758	20300911
107758	20300912
107758	20300913
107758	20300914
107758	20300915
107758	20300916
107758	20300918
107758	20300919
107758	20300920
107758	20300921
107758	20300922
107758	20300923
107758	20300924
107758	20300925
107758	20300936
107758	20300937
107758	20300938
107758	20300939
107758	20300940
107758	20300941
107758	20300942
107758	20300943
107758	20300944
107758	20307724
107758	20307725
107758	20307726
107758	20307727
107758	20307728
107758	20307729
107758	20307730
107758	20307731
107758	20307732
107758	20307733
107758	20307744
107758	20307745
107758	20307746
107758	20307747
107758	20307748
107758	20307750
107758	20307751

MPU Model Number	MPU Serial Number
107758	20307752
107758	20307754
107758	20307756
107758	20307757
107758	20307758
107758	20307760
107758	20307761
107758	20307762
107758	20307763
107758	20307764
107758	20307765
107758	20307766
107758	20307767
107758	20307768
107758	20307769
107758	20307770
107758	20307771
107758	20307772
107758	20307773
107758	20307774
107758	20307775
107758	20307776
107758	20307777
107758	20307778
107758	20307779
107758	20307780
107758	20307781
107758	20307782
107758	20307783
107758	20307784
107758	20307785
107758	20307786
107758	20307787
107758	20307788
107758	20307789
107758	20307790
107758	20307791
107758	20307792
107758	20307793
107758	20307794
107758	20307795
107758	20307796
107758	20307797
107758	20307798
107758	20307799
107758	20307800

## Listing of Potentially Impacted MPUs

Model 107758, Serial Number Range: 20226183 - 20326942

MPU Model Number	MPU Serial Number
107758	20307801
107758	20307803
107758	20307804
107758	20307805
107758	20307806
107758	20307807
107758	20307808
107758	20307809
107758	20307810
107758	20307811
107758	20307812
107758	20307813
107758	20307814
107758	20307815
107758	20307816
107758	20307817
107758	20307819
107758	20307820
107758	20307821
107758	20307822
107758	20307823
107758	20307824
107758	20307825
107758	20307826
107758	20307827
107758	20307828
107758	20307829
107758	20307830
107758	20307831
107758	20307832
107758	20307833
107758	20307834
107758	20307835
107758	20307836
107758	20307837
107758	20307838
107758	20307839
107758	20307840
107758	20307841
107758	20307842
107758	20307843
107758	20326857
107758	20326859
107758	20326861
107758	20326863
107758	20326865

MPU Model Number	MPU Serial Number
107758	20326866
107758	20326867
107758	20326868
107758	20326869
107758	20326870
107758	20326871
107758	20326872
107758	20326873
107758	20326874
107758	20326875
107758	20326876
107758	20326877
107758	20326878
107758	20326879
107758	20326880
107758	20326881
107758	20326883
107758	20326884
107758	20326915
107758	20326916
107758	20326917
107758	20326918
107758	20326919
107758	20326920
107758	20326921
107758	20326922
107758	20326923
107758	20326924
107758	20326935
107758	20326936
107758	20326937
107758	20326938
107758	20326939
107758	20326940
107758	20326941
107758	20326942

### Listing of Potentially Impacted MPUs

Model 107758AU, Serial Number Range: 20314704 - 20314713

MPU Model Number	MPU Serial Number
107758AU	20314704
107758AU	20314705
107758AU	20314706
107758AU	20314707
107758AU	20314709

MPU Model Number	MPU Serial Number
107758AU	20314710
107758AU	20314711
107758AU	20314712
107758AU	20314713

## Listing of Potentially Impacted MPUs

Model 107758UK, Serial Number Range: 20242028 - 20307743

MPU Model Number	MPU Serial Number
107758UK	20242028
107758UK	20242029
107758UK	20242030
107758UK	20242031
107758UK	20242032
107758UK	20242033
107758UK	20242034
107758UK	20242035
107758UK	20242036
107758UK	20242037
107758UK	20254073
107758UK	20254074
107758UK	20254075
107758UK	20254076
107758UK	20254077
107758UK	20254078
107758UK	20254079
107758UK	20254080
107758UK	20254081
107758UK	20254082
107758UK	20254113
107758UK	20254114
107758UK	20254115

MPU Model Number	MPU Serial Number
107758UK	20254116
107758UK	20254117
107758UK	20254118
107758UK	20254119
107758UK	20254120
107758UK	20254121
107758UK	20254122
107758UK	20300926
107758UK	20300927
107758UK	20300928
107758UK	20300929
107758UK	20300930
107758UK	20300931
107758UK	20300932
107758UK	20300933
107758UK	20300934
107758UK	20300935
107758UK	20307734
107758UK	20307735
107758UK	20307736
107758UK	20307737
107758UK	20307743



### Listing of Potentially Impacted MPUs

Model L107758, Serial Number Range: 20254089 - 20296534

MPU Model Number	MPU Serial Number
<b>L107758</b>	20254089
<b>L107758</b>	20277239

MPU Model Number	MPU Serial Number
<b>L107758</b>	20296525
<b>L107758</b>	20296534