



Urgent Field Safety Notice

Heart Failure Division
Thoratec Switzerland GmbH
Technoparkstrasse 1
CH-8005 Zürich Switzerland

FA-Q224-HF-2

2nd Generation CentriMag Primary Console
(Model Number: 201-30300, 201-90401, 201-90701, L201-90401, L201-90421)

May 2024

Dear Valued Customer,

Abbott is notifying users of a potential issue identified through our internal process with the **CentriMag Console (2nd Generation CentriMag Primary Console)** which is a part of the CentriMag™ Circulatory Support System. Internal testing showed that due to a component change, the CentriMag Console may not conform to the IEC 61000-4-5 standard. If subjected to a power surge above 1.8kV and up to 2.0 kV, the CentriMag Console may shut down completely without alarm. A power surge may be caused by, including, but not limited to the following: lightning strike, unregulated power switching, or faulty equipment. Pumping would cease and the patient would have to be switched to a backup.

There have been no reported complaints for distributed products and no reports of patient harm or adverse events due to this issue. Only surge voltage above 1.8 kV may shut down the CentriMag console without alarm. The ability of the user to transfer to a backup console to restore system function is not impacted. The affected product is not being removed from the field and does not need to be returned.

This letter contains important information and recommendations on how to use CentriMag Console if the system shuts down without an alarm.

Abbott discovered that certain CentriMag Console manufactured since March 21, 2016 are impacted by this issue. Please review Appendix A for the specific model and serial numbers within the scope of this notification.

Impact and Associated Risks

When affected CentriMag Console is exposed to surge voltage above 1.8 kV and up to 2.0 kV, if the Console shuts down without alarming, resulting impact can be delay in hemodynamic support while support is transferred to the backup system, or potentially thromboembolism if patient support is not transferred in a timely manner.

Recommendation

If the CentriMag Console shuts down without alarms or alerts present, follow Abbott Instruction for Use (IFU) for *Console and Motor malfunction* as described in the Section “Emergency / Troubleshooting”, and switch to a backup CentriMag Console which must be prepared as required backup components in the immediate vicinity.

Note: When the backup CentriMag console is put into use, the previously used console can be restarted and set up as a backup console.

Per the IFU and current practice, a full CentriMag Console backup system is always required to be available in the immediate vicinity of each patient whenever the CentriMag system is used.



Abbott

Urgent Field Safety Notice

Heart Failure Division
Thoratec Switzerland GmbH
Technoparkstrasse 1
CH-8005 Zürich Switzerland

FA-Q224-HF-2

2nd Generation CentriMag Primary Console
(Model Number: 201-30300, 201-90401, 201-90701, L201-90401, L201-90421)

Abbott performed additional testing and confirmed that the ability to pump and alarm is not affected in backup systems. As stated in the IFU these backup systems should have a battery charge sufficient for at least one hour of operation, be connected to wall power, and be powered off.

Abbott is in the process of implementing an additional in-process test by Q3 of 2024, which will ensure that all systems which will be distributed are confirmed to withstand a surge up to 2.0 kV.

Please distribute this notice to those who need to be aware within your institution and to any organization where potentially affected devices may have been transferred in the event that devices were transferred elsewhere.

Abbott has notified applicable regulatory agencies about this issue. Adverse reactions or quality problems experienced with the use of this product may be reported to your local Abbott representative.

Abbott is committed to providing the highest quality products and support, and we thank you for your partnership in assisting us with this process. Should you have any questions about this communication, please contact your local Abbott representative.

Sincerely,

A handwritten signature in blue ink, appearing to read 'Dan Lack'.

Dan Lack
Senior Manager Quality
Quality Management Representative
Abbott Heart Failure

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	20076173
201-90401	20079884
201-90401	19369099
201-90401	19383628
201-90401	L07174-0001
201-90401	L07470-0001
L201-90401	L03281-0007
L201-90401	L03345-0007
201-90401	L06929-0003
201-90401	L06929-0010
201-30300	L06357-0001
201-90401	L01911-0008
201-90401	L01911-0009
201-90401	L02071-0005
201-90401	L02071-0006
201-90401	L06357-0003
201-90401	L06357-0004
201-90401	L06357-0008
201-90401	L06495-0001
201-90401	L06495-0002
201-90401	L06495-0007
201-90401	L06495-0008
201-90401	L06495-0009
201-90401	L06525-0005
201-90401	L06553-0001
201-90401	L06554-0002
201-90401	L06554-0004
201-90401	L06554-0006
201-90401	L06554-0007
201-90401	L06554-0009
201-90401	L06554-0010
201-90401	L06556-0004
201-90401	L06556-0005
201-90401	L06556-0006
201-90401	L06556-0007
201-90401	L06556-0009
201-90401	L06579-0001
201-90401	L06579-0002
201-90401	L06579-0005
201-90401	L06579-0006
201-90401	L06583-0003
201-90401	L06583-0009
201-90401	L06586-0005
201-90401	L06587-0004
201-90401	L06587-0005
201-90401	L06587-0006

Model	Serial
201-90401	L06932-0008
201-90401	L06940-0001
201-90401	L06949-0008
201-90401	L07030-0001
201-90401	L07050-0001
201-90401	L07059-0001
201-90401	L07205-0001
201-90401	L07245-0001
201-90401	L07606-0001
201-90401	L07612-0001
201-90401	L07627-0001
201-90401	L07822-0001
201-90401	L07929-0001
201-90701	L04306-0004
201-90701	L05689-0001
201-90701	L05689-0002
201-90701	L05776-0008
201-90701	L05853-0001
201-90701	L05853-0002
201-90701	L05853-0003
201-90701	L05853-0004
201-90701	L05853-0006
201-90701	L06338-0004
201-90701	L06492-0004
201-90701	L06557-0003
201-90701	L06580-0007
201-90701	L06757-0005
201-90701	L06757-0007
201-90701	L07986-0001
201-90401	L07376-0001
201-90701	L07308-0001
201-90401	L04648-0005
201-90401	L04648-0006
201-90401	L04648-0007
201-90401	L04648-0008
201-90401	L04648-0009
201-90401	L04734-0001
201-90401	L06946-0005
201-90401	L06946-0006
201-90701	L06941-0006
201-90701	L06941-0009
201-90401	19236360
201-90401	L06929-0008
201-90401	L06930-0005
201-90401	L06930-0006
201-90401	L06930-0010

Model	Serial
201-90401	L06388-0008
201-90401	L06404-0010
201-90401	L06405-0002
201-90401	L06405-0010
201-90401	L06406-0010
201-90401	L06407-0003
201-90401	L06407-0006
201-90401	L06407-0009
201-90401	L06472-0008
201-90401	L06487-0010
201-90401	L06489-0004
201-90401	L06489-0005
201-90401	L06489-0006
201-90401	L06489-0009
201-90401	L06494-0001
201-90401	L06494-0002
201-90401	L06494-0005
201-90401	L06528-0005
201-90401	L06530-0004
201-90401	L06554-0003
201-90401	L06582-0001
201-90401	L06583-0001
201-90401	L06583-0002
201-90401	L06583-0004
201-90401	L06583-0007
201-90401	L06587-0010
201-90401	L06591-0008
201-90401	L06591-0009
201-90401	L06595-0001
201-90401	L06596-0002
201-90401	L06618-0002
201-90401	L06618-0005
201-90401	L06618-0007
201-90401	L06626-0006
201-90401	L06753-0007
201-90401	L06753-0008
201-90401	L06753-0009
201-90401	L06758-0010
201-90401	L06776-0005
201-90401	L06780-0005
201-90401	L06780-0007
201-90401	L06780-0009
201-90401	L06784-0001
201-90401	L06784-0006
201-90401	L06785-0006
201-90401	L06785-0008

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	L06596-0003
201-90401	L06596-0004
201-90401	L06596-0005
201-90401	L06596-0006
201-90401	L06596-0007
201-90401	L06596-0008
201-90401	L06596-0009
201-90401	L06596-0010
201-90401	L06624-0008
201-90401	L06754-0001
201-90401	L06754-0002
201-90401	L06754-0005
201-90401	L06754-0007
201-90401	L06754-0010
201-90401	L06801-0007
201-90401	L06866-0001
201-90401	L06866-0004
201-90401	L06876-0009
201-90401	L06932-0001
201-90401	L06932-0003
201-90401	L06932-0009
201-90401	L06932-0010
201-90401	L06306-0004
201-90401	L06306-0005
201-90401	L06306-0006
201-90401	L06402-0001
201-90401	L06402-0004
201-90401	L06402-0005
201-90401	L06402-0007
201-90701	L05831-0005
201-90701	L05831-0006
201-90701	L05831-0007
201-90701	L05831-0008
L201-90401	L05437-0003
201-90401	19236356
201-90401	19236362
201-30300	L03111-0008
201-30300	L03378-0005
201-90401	L03378-0004
201-90401	L04556-0005
201-90401	L04579-0001
201-90401	L06066-0010
201-90401	L06298-0003
201-90401	L06406-0007
201-90401	L06406-0008
201-90401	L06785-0001

Model	Serial
201-90401	L06934-0001
201-90401	19250293
201-90401	19250297
201-90401	L06784-0002
201-90401	L06786-0004
201-90401	L06786-0005
201-90401	19326654
201-90401	19400420
201-90401	L01053-0007
201-90401	L01479-0009
201-90401	L06554-0005
201-90401	L06554-0008
201-90401	L06758-0001
201-90401	L06758-0002
201-90401	L06758-0003
201-90401	L06758-0004
201-90401	L06758-0005
201-90401	L06758-0006
201-90401	L06758-0007
201-90401	L06758-0008
201-90401	L06776-0002
201-90401	L06776-0004
201-90401	L06778-0004
201-90401	L06785-0003
201-90401	L07073-0001
201-90401	L07074-0001
201-90401	L06587-0001
201-90401	L07386-0001
201-90401	L07388-0001
201-90401	L07421-0001
201-90701	L07327-0001
201-90701	L07330-0001
201-30300	L06588-0004
201-90401	L06588-0002
201-90401	L06588-0005
201-90401	L06588-0006
201-90401	L06588-0007
201-90401	L06588-0008
201-90401	L06754-0004
201-90401	L06754-0006
201-30300	L02080-0008
201-90401	L04579-0008
201-90401	L04594-0010
201-90401	L04913-0002
201-90401	L04913-0008
201-90401	L04923-0006

Model	Serial
201-90401	L06863-0001
201-90401	L06866-0002
201-90401	L06875-0001
201-90401	L06875-0006
201-90401	L06875-0007
201-90401	L06876-0001
201-90401	L06921-0001
201-90401	L06923-0001
201-90401	L06923-0009
201-90401	L06924-0001
201-90401	L06924-0002
201-90401	L06938-0008
201-90401	L06938-0010
201-90401	L06949-0001
201-90401	L07039-0001
201-90401	L07072-0001
201-90401	L07075-0001
201-90401	L07076-0001
201-90401	L07149-0001
201-90401	L07156-0001
201-90401	L07173-0001
201-90401	L07210-0001
201-90401	L07225-0001
201-90401	L07263-0001
201-90401	L07356-0001
201-90401	L07368-0001
201-90401	L07382-0001
201-90401	L07944-0001
201-90401	L07988-0001
201-90701	L07972-0001
L201-90401	L04096-0003
L201-90401	L04100-0010
L201-90401	L04503-0010
L201-90401	L04673-0005
L201-90401	L04868-0001
L201-90401	L201-90401
L201-90421	L05570-0019
201-30300	L02312-0009
201-30300	L04549-0002
201-30300	L06786-0002
201-90401	19383631
201-90401	19400414
201-90401	19410763
201-90401	19415571
201-90401	20119750
201-90401	L01246-0010

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	L06785-0002
201-90401	L06804-0003
201-90401	L06804-0006
201-90401	L06875-0008
201-90401	L06875-0009
201-90401	L07487-0001
201-90701	L05005-0006
201-90701	L06338-0003
201-90701	L07333-0001
201-90701	L07335-0001
201-90401	19450850
201-90401	19923331
201-90401	20060493
201-90401	20076101
201-90401	L01053-0002
201-90401	L04487-0007
201-90401	L04487-0008
201-90401	L04776-0009
201-90401	L04868-0003
201-90401	L04900-0008
201-90401	L04966-0006
201-90401	L04966-0007
201-90401	L06592-0010
201-90401	L06617-0009
201-90401	L06776-0003
201-90401	L06776-0008
201-90401	L06776-0009
201-90401	L06778-0002
201-90401	L06778-0009
201-90401	L06780-0006
201-90401	L06784-0010
201-90401	L06786-0003
201-90401	L06876-0006
201-90401	L06876-0008
201-90401	L07372-0001
201-90401	L07471-0001
201-90401	L07480-0001
201-90401	L07542-0001
201-90401	L07611-0001
201-90701	L06492-0003
201-90701	L07331-0001
201-90701	L07332-0001
201-30300	L03983-0004
201-90401	19236354
201-90401	19236357
201-90401	19421180

Model	Serial
201-90401	L04942-0002
201-90401	L04948-0008
201-90401	L05238-0005
201-90401	L05238-0006
201-90401	L06586-0009
201-90401	L06587-0003
201-90401	L06587-0007
201-90401	L06590-0005
201-90401	L06591-0010
201-90401	L06596-0001
201-90401	L06783-0003
201-90401	L06784-0005
201-90401	L06784-0007
201-90401	L06784-0008
201-90701	L06494-0009
201-90701	L06494-0010
201-90701	L06579-0008
201-90701	L06579-0010
201-90701	L06580-0004
201-90701	L06757-0001
201-90401	L04900-0006
201-90401	L04900-0007
201-90401	L05033-0007
201-90401	L05033-0008
201-90401	L06308-0010
201-90401	L06358-0002
201-90401	L06358-0009
201-90401	L06358-0010
201-90401	L06402-0008
201-90401	L06530-0002
201-90401	L06595-0002
201-90401	L06595-0003
201-90401	L06753-0002
201-90401	L06753-0003
201-90401	L06780-0001
201-90401	L06780-0002
201-90401	L06780-0003
201-90401	L06780-0004
201-90401	L07182-0001
201-90401	L07188-0001
L201-90421	L05333-0011
201-90401	L06337-0006
201-90401	L06358-0006
201-90401	L06526-0004
201-90401	L06526-0009
201-90401	L06773-0001

Model	Serial
201-90401	L01705-0008
201-90401	L02268-0006
201-90401	L04549-0001
201-90401	L04549-0003
201-90401	L04549-0004
201-90401	L04753-0002
201-90401	L04776-0004
201-90401	L05238-0010
201-90401	L05241-0001
201-90401	L06357-0005
201-90401	L06357-0009
201-90701	L05005-0007
201-90701	L05005-0009
201-90701	L05776-0006
201-90701	L05776-0007
201-90701	L06757-0003
201-90701	L06757-0010
201-30300	L03552-0003
201-30300	L03552-0004
201-30300	L03552-0006
201-90401	19942146
201-90401	19944576
201-90401	19950808
201-90401	19955812
201-90401	19955813
201-90401	19969064
201-90401	L03543-0008
201-90401	L03552-0005
201-90401	L03748-0008
201-90401	L03774-0004
201-90401	L03983-0008
201-90401	L06337-0001
201-90401	L06358-0003
201-90401	L06358-0004
201-90401	L06358-0005
201-90401	L06358-0007
201-90401	L06365-0002
201-90401	L06404-0003
201-90401	L06404-0009
201-90401	L06407-0007
201-90401	L06473-0006
201-90401	L06489-0001
201-90401	L06595-0005
201-90401	L06595-0006
201-90401	L06595-0007
201-90401	L06595-0010

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	19421181
201-90401	19421298
201-90401	19421299
201-90401	19434057
201-90401	19445268
201-90401	19507821
201-90401	19520134
201-90401	20001688
201-90401	20040697
201-90401	L00610-0011
201-90401	L03117-0006
201-90401	L04096-0004
201-90401	L04512-0009
201-90401	L04594-0001
201-90401	L04594-0009
201-90401	L04913-0007
201-90401	L04942-0001
201-90401	L05682-0001
201-90401	L05682-0002
201-90401	L06299-0002
201-90401	L06299-0005
201-90401	L06309-0009
201-90401	L06558-0006
201-90401	L06580-0006
201-90401	L06616-0002
201-90401	L06617-0003
201-90401	L06627-0005
201-90401	L06775-0007
201-90401	L06783-0001
201-90401	L06783-0006
201-90401	L06783-0009
201-90401	L06783-0010
201-90401	L06875-0002
201-90401	L06876-0003
201-90401	L06923-0004
201-90401	L06923-0005
201-90401	L06923-0006
201-90401	L06923-0007
201-90401	L06923-0008
201-90401	L07213-0001
201-90401	L07224-0001
201-90401	L07232-0001
201-90401	L07233-0001
201-90401	L07235-0001
201-90401	L07237-0001
201-90401	L07239-0001

Model	Serial
201-90401	L06773-0002
201-90401	L06930-0007
201-90401	L06930-0008
201-90401	L06945-0003
201-90401	L06945-0009
201-90401	L06060-0004
201-90401	L06492-0009
201-90401	19336026
201-90401	19415573
201-90401	L04948-0001
201-90401	L05308-0003
201-90401	L05308-0004
201-90401	L05308-0005
201-90401	L05308-0008
201-90401	L05308-0010
201-90401	L05326-0001
201-90401	L05427-0009
201-90401	L05427-0010
201-90401	L05437-0002
201-90401	L05437-0004
201-90401	L05437-0006
201-90401	L05437-0007
201-90401	L05556-0001
201-90401	L05556-0002
201-90401	L06588-0003
201-90401	L06588-0009
201-90401	L06786-0006
201-90401	L06786-0007
201-90401	L06786-0008
201-90401	L06786-0009
201-90401	L06786-0010
201-90401	L06801-0010
201-90401	L06804-0005
201-90401	L06805-0003
201-90401	L06805-0004
201-90401	L06805-0005
201-90401	L06805-0007
201-90401	L06805-0010
201-90401	L06863-0002
201-90401	L06863-0003
201-90401	L06863-0004
201-90401	L06863-0007
201-90401	L06866-0003
201-90401	L06866-0005
201-90401	L06866-0006
201-90401	L06866-0009

Model	Serial
201-90401	L06618-0001
201-90401	L06618-0009
201-90401	L06627-0004
201-90401	L06627-0007
201-90401	L06627-0008
201-90401	L06783-0005
201-90401	L06783-0007
201-90401	L06783-0008
201-90401	L06785-0004
201-90401	L06785-0005
201-90401	L06785-0007
201-90401	L06785-0009
201-90401	L06799-0004
201-90401	L06799-0005
201-90401	L06799-0008
201-90401	L06799-0010
201-90401	19259724
201-90401	19259725
201-90401	19260460
201-90401	L06595-0004
201-90401	L06618-0003
201-90701	L07930-0001
201-90701	L07931-0001
201-90701	L07933-0001
201-90401	L05869-0004
201-90401	L05869-0007
201-90701	L07009-0001
201-90701	L07010-0001
201-90701	L07013-0001
201-90701	L07153-0001
201-30300	L01479-0010
201-30300	L01868-0006
201-30300	L03059-0002
201-30300	L03345-0008
201-30300	L03748-0004
201-30300	L03748-0010
201-30300	L03930-0003
201-30300	L04354-0006
201-30300	L04406-0007
201-30300	L04406-0010
201-30300	L06774-0001
201-90401	19234130
201-90401	19236353
201-90401	19236361
201-90401	19250288
201-90401	19250289

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	L07240-0001
201-90401	L07241-0001
201-90401	L07243-0001
201-90401	L07246-0001
201-90401	L07248-0001
201-90401	L07352-0001
201-90401	L07358-0001
201-90401	L07359-0001
201-90401	L07361-0001
201-90401	L07362-0001
201-90401	L07363-0001
201-90401	L07364-0001
201-90401	L07369-0001
201-90401	L07378-0001
201-90401	L07633-0001
201-90401	L07644-0001
201-90401	L07650-0001
201-90401	L07651-0001
201-90401	L07948-0001
201-90701	L07015-0001
L201-90401	L00694-0006
L201-90401	L01053-0003
L201-90401	L01458-0004
L201-90401	L01705-0002
L201-90401	L02071-0001
L201-90401	L02101-0009
L201-90401	L02101-0010
L201-90401	L02230-0003
L201-90401	L02230-0004
L201-90401	L03000-0003
L201-90401	L03082-0003
L201-90401	L03082-0006
L201-90401	L03117-0005
L201-90401	L03117-0009
L201-90401	L03364-0002
L201-90401	L03543-0009
L201-90401	L03552-0002
L201-90401	L03774-0005
L201-90401	L04267-0010
L201-90401	L04286-0004
L201-90401	L04286-0005
L201-90401	L04468-0001
L201-90401	L04468-0007
L201-90401	L04468-0008
L201-90401	L04468-0009
L201-90401	L04468-0010

Model	Serial
201-90401	L06876-0002
201-90401	L06876-0004
201-90401	L06876-0005
201-90401	L06903-0001
201-90401	L06903-0003
201-90401	L06903-0005
201-90401	L06903-0006
201-90401	L06903-0007
201-90401	L06903-0008
201-90401	L06903-0009
201-90401	L06903-0010
201-90401	L06905-0001
201-90401	L06905-0002
201-90401	L06905-0003
201-90401	L06905-0004
201-90401	L06905-0005
201-90401	L06905-0006
201-90401	L06905-0007
201-90401	L06905-0008
201-90401	L06905-0009
201-90401	L06905-0010
201-90401	L06906-0002
201-90401	L06906-0003
201-90401	L06906-0005
201-90401	L06906-0006
201-90401	L06906-0008
201-90401	L06906-0009
201-90401	L06906-0010
201-90401	L06933-0005
201-90401	L06933-0006
201-90401	L06933-0008
201-90401	L06933-0009
201-90401	L06933-0010
201-90401	L06944-0001
201-90401	L06944-0002
201-90401	L06944-0003
201-90401	L06944-0010
201-90401	L06945-0005
201-90401	L06945-0006
201-90701	L03774-0007
201-90701	L03774-0008
201-90701	L04306-0005
201-90701	L04487-0006
201-90701	L05005-0008
201-90701	L05526-0001
201-90701	L05526-0002

Model	Serial
201-90401	19250294
201-90401	19250295
201-90401	19326649
201-90401	19326651
201-90401	19326652
201-90401	19326653
201-90401	19361780
201-90401	19361782
201-90401	19369085
201-90401	19374584
201-90401	19379721
201-90401	19379723
201-90401	19379725
201-90401	19379727
201-90401	19421301
201-90401	19429365
201-90401	20001689
201-90401	20001690
201-90401	20078984
201-90401	20079886
201-90401	L00934-0007
201-90401	L00934-0009
201-90401	L01216-0003
201-90401	L01308-0007
201-90401	L01308-0008
201-90401	L01479-0006
201-90401	L01868-0007
201-90401	L01940-0001
201-90401	L03279-0009
201-90401	L03281-0010
201-90401	L03766-0010
201-90401	L03930-0004
201-90401	L04354-0005
201-90401	L04354-0007
201-90401	L04406-0006
201-90401	L04437-0010
201-90401	L04753-0005
201-90401	L04780-0009
201-90401	L05005-0004
201-90401	L05115-0006
201-90401	L05129-0007
201-90401	L05129-0008
201-90401	L05129-0009
201-90401	L05526-0010
201-90401	L05556-0006
201-90401	L05682-0006

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
L201-90401	L04487-0001
L201-90401	L04487-0002
L201-90401	L04487-0003
L201-90401	L04487-0004
L201-90401	L04487-0005
L201-90401	L04487-0009
L201-90401	L04487-0010
L201-90401	L04494-0001
L201-90401	L04494-0002
L201-90401	L04494-0003
L201-90401	L04494-0004
L201-90401	L04494-0005
L201-90401	L04494-0006
L201-90401	L05939-0010
L201-90401	L06405-0004
L201-90401	L06553-0003
L201-90401	L06579-0004
201-90401	L06580-0003
201-90401	L06580-0005
201-90401	L06585-0009
201-90401	L06586-0010
201-90401	L06597-0009
201-90401	L06866-0010
201-90401	L06903-0004
201-90401	L06921-0007
201-90401	L06925-0009
201-90401	L06928-0003
201-90401	L06932-0002
201-90401	L06934-0003
201-90401	L06935-0001
201-90401	L06949-0003
201-90401	L06949-0004
201-90401	L06949-0006
201-90401	L06949-0009
201-90401	L06949-0010
201-90401	L07029-0001
201-90401	L07031-0001
201-90401	L07035-0001
201-90401	L07048-0001
201-90401	L07049-0001
201-90401	L07270-0001
201-90401	L07272-0001
201-90401	L07279-0001
201-90401	L07280-0001
201-90401	L07281-0001
201-90401	L07283-0001

Model	Serial
201-90701	L05526-0003
201-90701	L05526-0004
201-90701	L05526-0005
201-90701	L05526-0006
201-90701	L05526-0007
201-90701	L05526-0008
201-90701	L05526-0009
201-90401	19478110
201-90401	20052392
201-90401	20052393
201-90401	L03389-0003
201-90701	19415572
201-90401	19236369
201-90401	19236371
201-90401	19250287
201-90401	19250298
201-90401	L06489-0010
201-90401	L06494-0004
201-90401	L06799-0007
201-90401	L06938-0003
201-90401	L07404-0001
201-90401	L05270-0001
201-90401	L05308-0006
201-90401	L05308-0007
201-90401	L05308-0009
201-90401	L06932-0004
201-90401	L06932-0005
201-90701	L06926-0001
201-90701	L06926-0002
201-90701	L06926-0003
201-90701	L06926-0004
201-90701	L06926-0005
201-90701	L06926-0006
201-90701	L06926-0008
201-90701	L06926-0010
201-30300	L01732-0005
201-30300	L01732-0006
201-30300	L02305-0004
201-30300	L02305-0006
201-90401	19236214
201-90401	19236365
201-90401	19250284
201-90401	19250286
201-90401	19379732
201-90401	19383629
201-90401	19400419

Model	Serial
201-90401	L05689-0007
201-90401	L05689-0008
201-90401	L05831-0009
201-90401	L05831-0010
201-90401	L05837-0005
201-90401	L05837-0006
201-90401	L05853-0007
201-90401	L05853-0008
201-90401	L05853-0009
201-90401	L05853-0010
201-90401	L05869-0001
201-90401	L05869-0002
201-90401	L05869-0003
201-90401	L05869-0005
201-90401	L05869-0006
201-90401	L05966-0005
201-90401	L06298-0004
201-90401	L06298-0005
201-90401	L06298-0006
201-90401	L06298-0007
201-90401	L06298-0010
201-90401	L06336-0003
201-90401	L06336-0004
201-90401	L06336-0005
201-90401	L06357-0006
201-90401	L06400-0001
201-90401	L06400-0002
201-90401	L06400-0003
201-90401	L06400-0004
201-90401	L06400-0005
201-90401	L06400-0006
201-90401	L06400-0007
201-90401	L06400-0008
201-90401	L06400-0009
201-90401	L06400-0010
201-90401	L06495-0003
201-90401	L06495-0004
201-90401	L06525-0009
201-90401	L06525-0010
201-90401	L06526-0001
201-90401	L06526-0002
201-90401	L06526-0005
201-90401	L06526-0006
201-90401	L06528-0002
201-90401	L06528-0003
201-90401	L06528-0004

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	L07284-0001
201-90401	L07288-0001
201-90401	L07289-0001
201-90401	L07290-0001
201-90401	L07292-0001
201-90401	L07350-0001
201-90401	L07351-0001
201-90401	L07353-0001
201-90401	L07354-0001
201-90401	L07355-0001
201-90401	L07365-0001
201-90401	L07366-0001
201-90401	L07371-0001
201-90401	L07373-0001
201-90401	L07380-0001
201-90401	L07383-0001
201-90401	L07384-0001
201-90401	L07392-0001
201-90401	L07394-0001
201-90401	L07399-0001
201-90401	L07413-0001
201-90401	L07414-0001
201-90401	L07419-0001
201-90401	L07420-0001
201-90701	L07157-0001
201-90701	L07158-0001
201-90701	L07255-0001
201-90701	L07305-0001
201-90701	L07334-0001
201-90701	L07401-0001
201-90701	L07403-0001
201-90401	19236363
201-90401	19236364
201-90401	20078985
201-90401	20079885
201-90401	L06930-0009
201-90401	L08005-0001
201-90401	L06357-0007
201-90401	L06357-0010
201-90401	19234129
201-90401	19234132
201-90401	19236209
201-90401	19236210
201-90401	19236212
201-90401	19250290
201-90401	19258709

Model	Serial
201-90401	19403747
201-90401	19406333
201-90401	19407984
201-90401	19410764
201-90401	19410765
201-90401	19430951
201-90401	19434051
201-90401	19445263
201-90401	19445266
201-90401	19450859
201-90401	19476965
201-90401	19604014
201-90401	20040698
201-90401	20116582
201-90401	20119751
201-90401	L00694-0003
201-90401	L00694-0004
201-90401	L01744-0002
201-90401	L01744-0003
201-90401	L02324-0008
201-90401	L02324-0010
201-90401	L03281-0004
201-90401	L03281-0005
201-90401	L03714-0001
201-90401	L03714-0007
201-90401	L03714-0009
201-90401	L03748-0002
201-90401	L04503-0007
201-90401	L04503-0008
201-90401	L04512-0001
201-90401	L04512-0002
201-90401	L04512-0003
201-90401	L04512-0004
201-90401	L04512-0005
201-90401	L04556-0004
201-90401	L04556-0006
201-90401	L04556-0007
201-90401	L04579-0002
201-90401	L04579-0003
201-90401	L04579-0004
201-90401	L04579-0005
201-90401	L04579-0010
201-90401	L04673-0001
201-90401	L04673-0003
201-90401	L04673-0004
201-90401	L04753-0003

Model	Serial
201-90401	L06528-0006
201-90401	L06528-0007
201-90401	L06528-0008
201-90401	L06590-0004
201-90401	L06626-0002
201-90401	L06626-0003
201-90401	L06626-0008
201-90401	L06626-0009
201-90401	L06627-0010
201-90401	L06754-0008
201-90401	L06773-0003
201-90401	L06773-0004
201-90401	L06773-0005
201-90401	L06773-0006
201-90401	L06773-0008
201-90401	L06773-0009
201-90401	L06773-0010
201-90401	L06776-0001
201-90401	L06776-0010
201-90401	L06777-0004
201-90401	L06778-0003
201-90401	L06778-0005
201-90401	L06778-0007
201-90401	L06778-0008
201-90401	L06780-0008
201-90401	L06781-0004
201-90401	L06781-0005
201-90401	L06781-0007
201-90401	L06781-0008
201-90401	L06781-0009
201-90401	L06781-0010
201-90401	L06801-0001
201-90401	L06801-0002
201-90401	L06801-0004
201-90401	L06801-0005
201-90401	L06801-0006
201-90401	L06804-0001
201-90401	L06804-0002
201-90401	L06804-0007
201-90401	L06804-0008
201-90401	L06804-0010
201-90401	L06805-0002
201-90401	L06863-0006
201-90401	L06863-0008
201-90401	L06866-0007
201-90401	L06866-0008

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	19258992
201-90401	19372720
201-90401	19376300
201-90401	19383633
201-90401	19406332
201-90401	L04295-0004
201-90401	L04295-0007
201-90401	L04295-0008
201-90401	L04295-0010
201-90401	L04556-0009
201-90401	L04556-0010
201-90401	L04868-0004
201-90401	L04868-0007
201-90401	L04900-0010
201-90401	L04948-0009
201-90401	L05326-0002
201-90401	L05556-0005
201-90401	L05569-0001
201-90401	L05569-0002
201-90401	L06298-0002
201-90401	L06337-0002
201-90401	L06337-0010
201-90401	L06364-0008
201-90401	L06402-0006
201-90401	L06404-0001
201-90401	L06404-0004
201-90401	L06405-0006
201-90401	L06405-0008
201-90401	L06492-0007
201-90401	L06492-0008
201-90401	L06492-0010
201-90401	L06530-0001
201-90401	L06530-0006
201-90401	L06530-0007
201-90401	L06530-0009
201-90401	L06556-0010
201-90401	L06778-0001
201-90401	L06778-0010
201-90401	L06781-0006
201-90401	L06783-0002
201-90401	L06783-0004
201-90401	L06799-0001
201-90401	L06799-0002
201-90401	L06799-0003
201-90401	L06799-0006
201-90401	L06799-0009

Model	Serial
201-90401	L04753-0004
201-90401	L04776-0007
201-90401	L04780-0008
201-90401	L04868-0006
201-90401	L04868-0008
201-90401	L04868-0009
201-90401	L04884-0001
201-90401	L04884-0003
201-90401	L04884-0006
201-90401	L04884-0007
201-90401	L04913-0009
201-90401	L04913-0010
201-90401	L04923-0003
201-90401	L04923-0004
201-90401	L04923-0005
201-90401	L04948-0002
201-90401	L04948-0003
201-90401	L04948-0007
201-90401	L04948-0010
201-90401	L05033-0006
201-90401	L05238-0004
201-90401	L05238-0007
201-90401	L05254-0006
201-90401	L05437-0001
201-90401	L05437-0005
201-90401	L05556-0009
201-90401	L05556-0010
201-90401	L05654-0006
201-90401	L05654-0008
201-90401	L05654-0009
201-90401	L05682-0008
201-90401	L05682-0010
201-90401	L05689-0009
201-90401	L05917-0001
201-90401	L05917-0009
201-90401	L05917-0010
201-90401	L05939-0002
201-90401	L05939-0004
201-90401	L05939-0008
201-90401	L05939-0009
201-90401	L05963-0001
201-90401	L06060-0001
201-90401	L06060-0002
201-90401	L06060-0005
201-90401	L06060-0007
201-90401	L06306-0009

Model	Serial
201-90401	L06875-0003
201-90401	L06875-0005
201-90401	L06875-0010
201-90401	L06906-0001
201-90401	L06906-0004
201-90401	L06906-0007
201-90401	L06921-0002
201-90401	L06921-0003
201-90401	L06921-0004
201-90401	L06921-0005
201-90401	L06921-0006
201-90401	L06921-0008
201-90401	L06921-0009
201-90401	L06921-0010
201-90401	L06924-0003
201-90401	L06924-0005
201-90401	L06924-0006
201-90401	L06924-0007
201-90401	L06924-0008
201-90401	L06924-0009
201-90401	L06924-0010
201-90401	L06927-0001
201-90401	L06927-0003
201-90401	L06927-0004
201-90401	L06927-0005
201-90401	L06927-0007
201-90401	L06927-0008
201-90401	L06928-0001
201-90401	L06928-0002
201-90401	L06928-0006
201-90401	L06928-0007
201-90401	L06928-0008
201-90401	L06928-0009
201-90401	L06928-0010
201-90401	L06935-0003
201-90401	L06935-0005
201-90401	L06935-0006
201-90401	L06935-0007
201-90401	L06935-0008
201-90401	L06935-0009
201-90401	L06935-0010
201-90401	L06937-0001
201-90401	L06937-0002
201-90401	L06937-0003
201-90401	L06939-0002
201-90401	L06939-0003

Appendix A: Specific Model and Serial Numbers Distributed in EMEA

Model	Serial
201-90401	L06804-0004
201-90401	L06804-0009
201-90401	L06923-0010
201-90401	L06925-0004
201-90401	L06927-0002
201-90401	L06932-0007
L201-90401	L06306-0008
201-90401	L06306-0010
201-90401	L06308-0005
201-90401	L06336-0001
201-90401	L06336-0002
201-90401	L06358-0001
201-90401	L06365-0008
201-90401	L06365-0009
201-90401	L06939-0005
201-90401	L06939-0006
201-90401	L06939-0009
201-90401	L06939-0010
201-90401	L07062-0001
201-90401	L07063-0001
201-90401	L07064-0001
201-90401	L07065-0001
201-90401	L07066-0001
201-90401	L07068-0001
201-90401	L07071-0001
201-90401	L07185-0001
201-90401	L07249-0001
201-90401	L07250-0001
201-90401	L07252-0001
201-90401	L07253-0001
201-90401	L07254-0001
201-90401	L07259-0001
201-90401	L07260-0001
201-90401	L07262-0001
201-90401	L07275-0001
201-90401	L07536-0001
201-90401	L07801-0001
201-90401	L07837-0001
201-90401	L07840-0001
201-90401	L07856-0001
201-90401	L07865-0001
201-90401	L07880-0001
201-90401	L07884-0001
201-90401	L07906-0001
201-90401	L07927-0001
201-90701	L05689-0003

Model	Serial
201-90701	L05689-0005
201-90701	L06757-0004
201-90701	L06757-0006
201-90701	L06757-0008
201-90701	L06757-0009
201-90701	L07985-0001
201-90701	L07992-0001
201-90701	L08004-0001
201-90701	L08007-0001
L201-90401	L04780-0007
L201-90401	L04884-0004
L201-90401	L04913-0003
L201-90401	L04913-0004
L201-90401	L04966-0001
L201-90401	L05033-0010
L201-90401	L05689-0006
L201-90401	L06306-0007